

# Unbreakable™ by tcs®

*The Modern Fit*



*For flexible partials with unbeatable strength*

- › An extremely resilient, high performance nylon thermoplastic
- › Guaranteed unbreakable due to its unique properties
- › Designed for exceptional memory to hold its form
- › Offers an ideal degree of flexibility
- › Stain resistant and biocompatible
- › A modern fit, lightweight and durable



**Monomer Free  
Hypoallergenic**



## INDICATIONS & BENEFITS

Economical, aesthetic solution for missing teeth  
Ideal for flexible partial dentures and unilaterals  
Metal-free solution  
Gingival veneers to mask exposed root surfaces  
Ideal for patients with allergies to monomer and/or nickel  
Transitional restoration during the healing period for implants



## SHADES



Light Pink



Dark Pink



Standard Pink



Natural



Light/Dark Pink



## INSURANCE CODES

D5225 Maxillary Partial Denture  
D5526 Mandibular Partial Denture



## TIPS

1. For best results use lightweight alginate impression materials.
2. Prior to initial insertion, soak the partial in hot tap water for approximately 1 minute. When the appliance cools to a tolerable temperature, gently insert it into the patient's mouth.
3. We recommend using the TCS Pink Zirconia Bur (5035Z) for minor adjustments and the TCS Brown Rubber Points (4513-01, 4514-01) to smooth the adjusted areas. Use light pressure in a scraping motion.
4. For a quick re-polish apply Green Hi-Shine polishing paste (4209-02) with a dry Mini Rag Buff (4221-01).
5. **Recommend the tcs® Concentrated Dental Appliance Cleaner to your patients for optimal appliance maintenance.**

*Chairside adjustment tools and FREE promo material available at [tcsdentalinc.com](http://tcsdentalinc.com).*

*"I love my TCS partial denture. It is comfortable, easy to care for,  
and best of all no one can tell I am wearing it."*

Specify Unbreakable™ by TCS on your Lab Slip

