


Try Fresh, each TCS case includes a 1-week supply sample sachet of the TCS Concentrated Dental Appliance Cleaner, patient instructions and a delivery bag.

*USA only

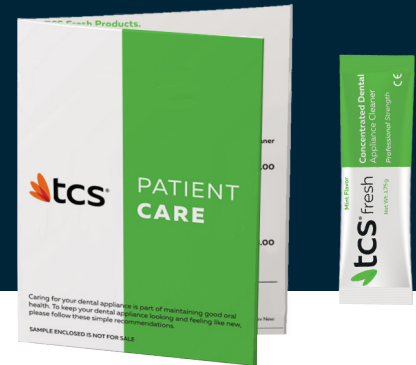

Proper Care

TCS appliances are very easy to care for when using TCS Fresh. In order to keep them looking and feeling like new, please have your patients follow these simple recommendations for daily cleaning:




1. Continue to brush your natural teeth and gums as directed.
2. Rinse the appliance under warm running water for approximately 10-30 seconds to remove any food particles.
3. Brush the appliance with a soft bristle brush using water only. Rinse again.
4. Create a cleansing solution with 1 packet of TCS Concentrated Dental Appliance Cleaner and about 1 cup of lukewarm water. Solution lasts an entire week (7 days).
5. Place the appliance in cleansing solution for at least 15 minutes daily and rinse the appliance thoroughly prior to re-insertion into mouth. When possible, keep appliance in water or the cleansing solution when not being worn.

IMPORTANT. Do not use bleach, toothpaste, or similar cleaning products, these may discolor or scratch the surface of the appliance causing micro-tears where bacteria can form in. Find a safe solution that works for patients.



For Happier Patients

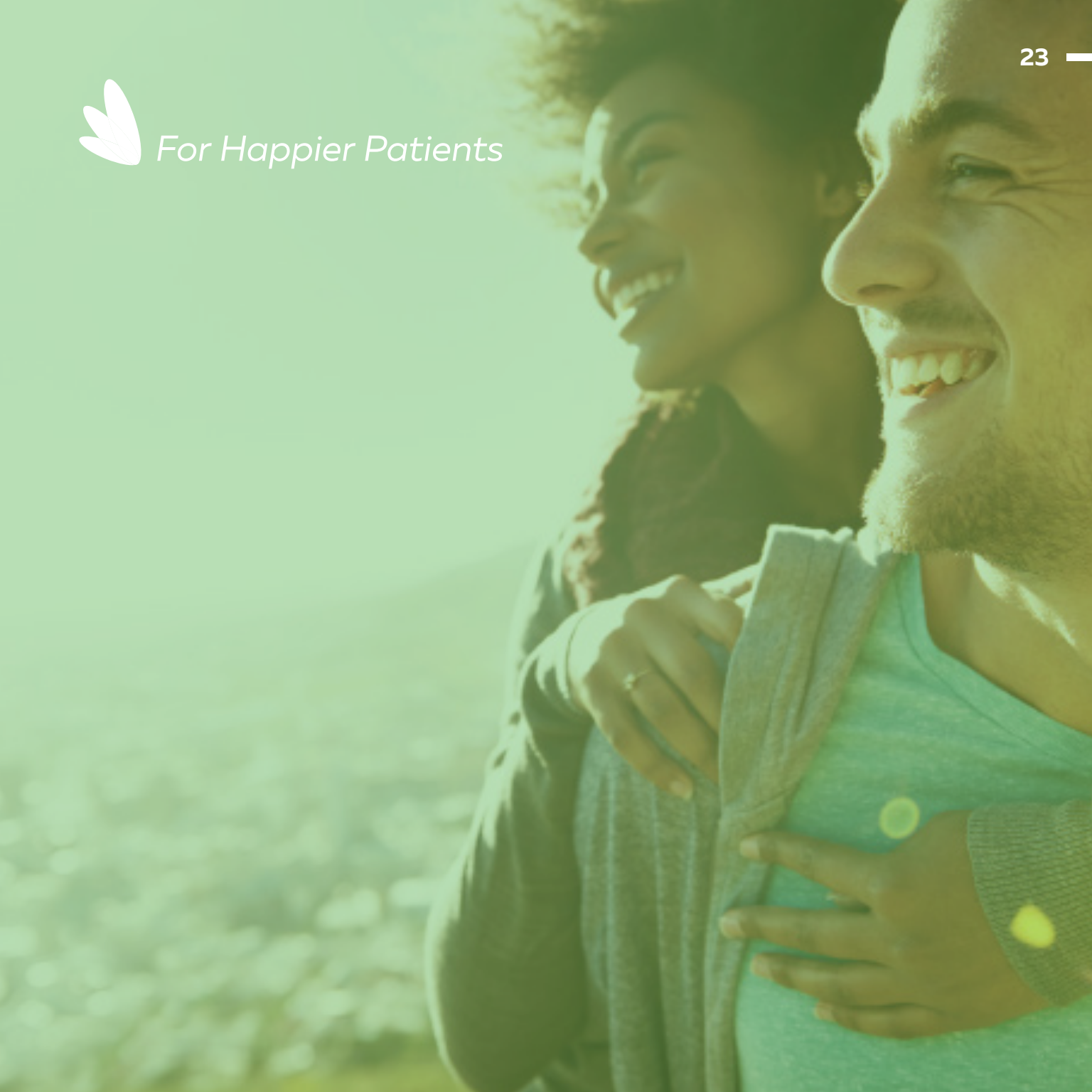



A clean appliance is vital to your patients' health and to the appearance and longevity of the appliance. TCS Concentrated Dental Appliance Cleaner is a safe solution for quick and effective results. Effectively cleans all types of removable dental appliances.

- Non-chlorinated solution (which means, it won't discolor, damage, or have an unpleasant taste and/or odor).
- Removes stains that brushing alone can leave behind.
E.g. Stains caused by tobacco, coffee, tea, blueberries, etc.
- Helps prevent iron stains as well as plaque, tartar, calcium and calculus.



Flexible Partial



Full Dentures



Metal Combo
Partials



Sportsguards
Snoreguards



Aligners
Nightguards



Retainers

TCS Fresh Products

Visit our website for discounted dental professional pricing and/or order free promotional material: www.tcsfresh.com



Ideal for
Dental
Offices



TCS Concentrated Dental Appliance Cleaner.

Professional strength cleanser. Deep cleans, removes stains and odors. Ideal for most types of removable dental appliances.

4505-02

24 pcs
6 month supply

TCS Dental Appliance Cleaner.

TCS Dental Appliance Cleaner sample packets. Box of 75 packets (1-week supply ea.).

4505FR-SMP

75 pcs

TCS Dental Appliance Brush.

Resilient soft bristle brush to thoroughly clean dental appliances without scratching the surface.

4506FR-01

FREE
DISPLAY WITH ORDER
OF 9+ BOXES

