



TCS Concentrated Dental Appliance Cleaner

Instructions of Use



1. Continue to brush your natural teeth and gums as directed by your dentist. Rinse your dental appliance thoroughly under warm water to remove any food particles.



2. Thoroughly brush your appliance with the TCS Dental Appliance brush using water only.




3. Stir entire contents of packet into approx. 1 cup of lukewarm water. Allow solution to stand for approx. 3 minutes.

Once prepared, solution lasts up to 7 days.



4. Submerge dental appliance in solution for at least 15 minutes daily. Extremely soiled appliances may require overnight soaking.

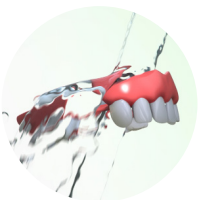

5. Rinse appliance thoroughly under running water prior to re-insertion into mouth.

Important. Dentures containing soft liners (flexible partials are not considered soft liners), aligners & mouthguards should not be soaked for more than 15 minutes per cleaning.

* Not recommended for use on prosthetics fabricated with Ticonium® Alloy.

Caution. Do not put powder or solution in mouth. Rinse well. Keep out of reach of children. Cleaner contains persulfates which may cause allergic reaction. If you experience irritation due to cleaner, discontinue use and consult a healthcare professional. In case of accidental ingestion, contact a Poison Control Center.