












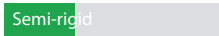



Choosing the Right TCS Resin

TCS offers three resilient, high performing resins for the fabrication of removable dental appliances. TCS resins have properties that allow removable appliances to be fabricated very thin without compromising durability and stability. Choose between three of the most comfortable and flexible resins on the market.

	Unbreakable™	iFlex™	Karadent™
Base material	Nylon thermoplastic	Polyolefin thermoplastic	Microcrystalline polymer
Flexibility			
Highly aesthetic			
Color stability			
Water absorption			
Easy to finish and polish	Technique sensitive	✓	✓
Durability	✓	✓	✓
BPA & monomer free	✓	✓	✓
Unbreakability	Guaranteed unbreakable	Clinically unbreakable	Clinically unbreakable
Translucency			
Clear shade	X	X	✓

Spruing 101



Take a Look

at the new Sprue Wax Guide on our Online Technical Library!

Learn the do's and don'ts of Sprue Waxing.

1. Use Sprue Wax with the right diameter (0.5 gauge)
2. Place your model with approx. 1.5 cm of separation from the saddles and the bottom of the flask
3. Use the TCS 9.5 Sprue Wax to prolong the life of your flask
4. Use no more than 2 -3 sprues
5. Connect your sprues with a hot spatula to melt and adhere the sprues to the model with a thin layer of wax

www.tcsdentalinc.com/technical-library/

On a more serious note
 TCS Goes Skydiving!



Members of the TCS team got together on a Saturday morning to go skydiving! Pictured is Larissa, our Marketing Assistant, jumping out of a perfectly good airplane.

BUY a Hydraulic or Automatic Basic Injector Kit

GET a \$100 Gift Card (choose from 10 retailers)
+ 2 Extra Packs (resin of your choice)
+ 1 Finishing and Polishing Tool Tester*.

BUY a Hydraulic or Automatic Complete Injector Kit

GET a \$250 Gift Card (choose from 10 retailers)
+ 4 Extra Packs (resin of your choice)
+ 1 Injection Flask + 1 TCS Apron.

BUY 10 packs

GET 1 Extra Pack (resin of choice)
+ 1 TCS Apron.



*Finishing and Polishing Tool Tester Includes: 1 Pink Zirconia Bur, 10 Brown Rubber Wheels, 4 Mini Fuzz Buffs, 3 Rag Buffs, and samples of TCS polishing pastes.

Must purchase from an authorized distributor. Limit 1 per customer. Promotions may not be combined. For end users only. Offer runs 05/10/2021 - 06/05/2021.

A Real Life TCS Case #MyTCS

A Teen Patient Receives a New TCS Partial Denture

Contrary to popular belief, dentures are not just for elderly patients. Patients of any age may lose some or all of their teeth and need an affordable, aesthetic solution. Here is a case by Protesis Dental Lab E. S. R. fabricated for a teenager who lost two upper teeth during oral surgery to remove a massive cyst. Due to the teen's age, medical experts decided to prescribe a removable dental appliance for two - three years to allow his mouth to mature. This well designed TCS appliance blends in naturally to the patient's surrounding tissue and gums. If seeing is believing, observe the images on the right to see how inconspicuous the appliance is in the patient's mouth.

Thank you Protesis Dental Lab E. S. R. for sharing this beautiful case!



Follow @tcsdentalinc Use #tcsdentalinc on instagram to share your recent TCS cases