


REBASE



iFlex™ & Unbreakable™ Rebase

Step-by-step guide

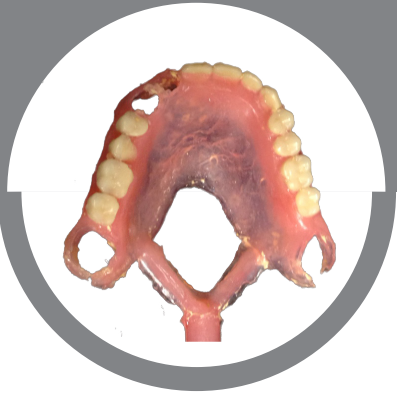

1. Take an alginate impression with the partial in the patients' mouth. Immediately pour up the model with stone.



2. Remove tray from the model.



3. Cut away excess impression material for preparation of bite registration index.



4. Take stone index.



5. Remove partial from the model and duplicate the model.

Pour Flask
Item # 3650-01
Duplicating Unit
Item # 3611-01 (110V)




6. Remove all teeth from existing partial.



7. Establish diatorics. Use Twist Drills for diatoric retention on teeth.
005- Item # 5005-01
006- Item # 5006-01
009- Item # 5009-01
012- Item # 5012-01



8. Secure teeth into stone index and wax accordingly.

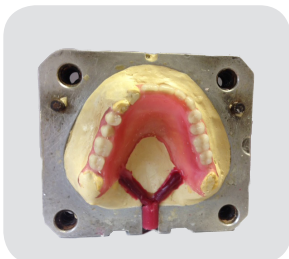

9. Sprue accordingly.
Injection Flask
Item # 3701-01
Sprue Wax (0.5 gauge)
Item # 5024-01 (1 lb)
Sprue 9.5 Wax
Item # 5027-01 (5 pc - yields 50 injections)



10. Boilout and apply separator.
EZ Flow Separator
Item # 4217-01 (16 oz)



11. Inject and proceed to finish and polish.

