EN THERMOPLASTIC DENTAL RESIN (Unbreakable™ by tcs®, Karadent™ by tcs®, iFlex™ by tcs®) CARTRIDGE, BULK, AND MILLABLE DISCS: FOR THE MANUFACTURE AND REPAIR OF REMOVABLE DENTAL PROSTHETICS INDICATED USE. Removable Partial Dentures and Complete Dentures including orthodontic devices, occlusal splints and night guards. **NOT SUITABLE FOR**. Fixed Restoration. **WARNINGS.** Patients should be instructed to clean their dental prosthesis daily to avoid bacterial and fungal (C.albicans) infections. Dental prosthesis are intended to be used by a single patient. Re-use of device for another patient may cause infection and device failure. Re-use of the dental resin or cartridge can result in mechanical failures of the molded prosthesis. Use eye protection during the milling and injection process. Use heat-resistant gloves during injection process. Use proper ventilation, vacuum system and mask during finishing stage.

Resin	Allergy Warning
Unbreakable™ by tcs® Karadent™ by tcs®	Should not be used for patients with an allergy to nylon.
iFlex™ by tcs®	Should not be used for patients with an allergy to polypropylene.

PRECAUTIONS. Patients may experience a "taste" disturbance upon initial use of dental prosthesis. **INSTRUCTIONS CARTRIDGE & BULK MODELS.**

- Once removed from sealed plastic bag, store in a dry place and at 90°F to 100°F (32°C to 38°C) at all times or at least 12 hours prior to use.
- Melt resin according to manufacturer’s instructions for type of furnace being utilized.

Approximate values for reference		
Resin	Temperature	Time
Unbreakable™ by tcs®	550°F (288°)	11 minutes
Karadent™ by tcs®	550°F - 600°F (288°C - 315°C)	15-17 minutes
iFlex™ by tcs®	447°F (230°C)	12.5 minutes

3. Proceed using standard methods and practices for thermoplastics. (Consult the Operating Instructions for the injection system you are using for detailed instructions).

INSTRUCTIONS MILLABLE DISC. Use standard methods and practices for milling thermoplastics. (Consult the operating manual for the milling system you are using for detailed instructions).

If any serious incident has occurred in relation to the device it should be reported to Thermoplastic Comfort Systems, Inc. (TCS) and to the competent authority of the Member State in which the user and/or patient is established.

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Izgatavots Thermoplastic Comfort Systems, Inc. 2019 Line Ave. Signal Hill, CA 90755 U.S.A. www.scientific.com
Fabricat de Thermoplastic Comfort Systems, Inc. 2019 Line Ave. Signal Hill, CA 90755 U.S.A. www.scientific.com