EN TCS Fresh – Concentrated Dental Appliance Cleaner

Intended Use. For cleaning retainers, aligners, mouthguards, snoreguards, sportsguards, nightguards, dentures, and partial dentures.
Instructions for Use: Rinse dental appliance thoroughly under running water. Optional: Brush dental appliance gently with a soft bristle denture brush. Stir entire contents of packet into approx. 1 cup of warm water. Allow solution to stand for approx. 3 minutes to fully dissolve powder.
Once prepared, solution lasts up to 7 days (solution should be used by one user for no more than 7 days to ensure the cleaning effectiveness of the solution). Submerge dental appliance in solution for at least 30 minutes daily. Rinse dental appliance thoroughly under running water prior to re-insertion into mouth. Extremely soiled appliances may require overnight soaking. Soak dental appliances daily for optimal results.
Important. Dentures containing soft liners (flexible partials are not considered soft liners), aligners and mouthguards should not be soaked for more than 15 minutes per cleaning.* Not recommended for use on prosthetics fabricated with Ticonium® Alloy.

⚠ Caution. Do not put powder or solution in mouth. Rinse well. Cleaner contains persulfates which may cause allergic reaction. If you experience irritation due to cleaner, discontinue use and consult a healthcare professional. In case of accidental ingestion, contact a Poison Control Center. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Do not use if packet has been opened or damaged.

🚫 Keep out of reach of children.

REF 4505, 4505-02, 4505-SLV.1	  	www.tcsfresh.com
Authorized European REP MDSG GmbH Schiffgraben 41 30175 Hannover, Germany	Manufactured by Thermoplastic Comfort Systems, Inc. 2619 Lime Ave. Signal Hill, CA 90756 U.S.A.	
EC REP		

REF 4505, 4505-02, 4505-SLV.1	  	www.tcsfresh.com
Representante autorizado en la Unión Europea MDSG GmbH Schiffgraben 41 30175 Hannover, Alemania	Fabricado por Thermoplastic Comfort Systems, Inc. 2619 Lime Ave. Signal Hill, CA 90756 U.S.A.	
EC REP		

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